dyras[®]

www.dyras.com

Copyright © 2013 The dyras wwe.factory LLC U.S.A.

No part of the content (including logos, design, photos, text, colour scheme, arrangements, etc.) may be reproduced, reused or redistributed for any purpose whatsoever, or distributed to a third party for such purposes, without the written permission of The dyras wwe.factory LLC.

* = registered trademark of The dyras wwe.factory LLC U.S.A

dyras[®]

BPSS-4129

Instruction manual EN

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Návod na použitie SK

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dyras BPSS-4129 wrist blood pressure meter user manual

NOTES

Please read the instruction manual thoroughly before using your new appliance. It is important to keep these instructions as a useful reminder that'll help you enjoy the many features available. The manufacturer is not responsible if the appliance becomes damaged or defective as a result of the user not following the information included in the instruction manual. If the appliance becomes faulty during regular use, it should only be repaired by an authorised service engineer.

IMPORTANT SAFETY INFORMATION

The following safety instructions must be followed when using the appliance.
Please read this section very carefully!

Of course, it's not possible for us to anticipate or provide safety rules for every specific situation, so we ask you to apply commonsense precautions when using the appliance in a situation that isn't described below.

- Never use the appliance outdoors.
- This device is not supposed to be used by physically or mentally handicapped people, or persons without experience or knowledge (including children, as well), unless such a person supervises or guides their use of the device, who is at the same time responsible for their safety. As for children, supervision is recommended to make sure that they do not play with the device. This device is not supposed to be used by children under the age of 14, not even with supervision!
- Never use an appliance or accessory that the manufacturer did not supply with or propose for use with the appliance. The use of such appliances might lead to accidents or irregular function.
- Don't put the appliance near hot objects (e.g. oven, hob, heating appliance etc.) or on any hot surface. Don't place the appliance anywhere it could slide down, fall or be knocked off accidentally.
- Don't use the appliance in the presence of explosive or inflammable gases, vapours or liquids.

Dear Customer,

All electrical and mechanical equipment needs to be cared for appropriately. It's important to note that using or storing the appliance in certain conditions may adversely affect its lifespan and functionality. Always protect the appliance from the conditions listed below.

Using or storing the appliance in a wet, humid or damp environment can cause:

- deterioration and premature aging of the cover and electric insulation of the appliance,
- premature wearing and aging of the plastic and rubber components,
- rust and corrosion affecting electrical components and metal parts.

The effects of direct sunlight on the appliance can include:

- accelerated deterioration of the appliance casing and its plastic and rubber components, damage to strength and other structural features. The appliance's surface may become porous and fragile, which may result in a safety risk.

Continuous heavy duty use may cause:

- premature structural wear and tear and premature aging of the motor due to extensive use of the appliance, resulting from wear and tear and overheating due to continuous use.

Storing the appliance in cold or frosty environment may cause:

- damage to the plastic and rubber components; if they become fragile, this may result in a safety risk.
- oxidation resulting from condensation, and corrosion of the metal components.

Direct and indirect heat (e.g. being too close to an oven or a heating appliance) can cause:

- damage, deformation, softening or melting of plastic or rubber parts and casing elements, or deterioration of electrical insulation.

Following the operating instructions and avoiding any of the conditions listed above will help you increase the lifespan of the appliance. The manufacturer is not responsible if the appliance is stored inappropriately or contrary to these instructions. You are recommended to have the appliance serviced regularly by an authorised engineer to keep it working for as long as possible.

- If your appliance is damaged or operates in an abnormal way, immediately stop using it and take it to an authorised service engineer for checking, repair or adjustment.
- The manufacturer is not responsible for any damage that results from overuse, incorrect operation, incorrect maintenance, neglecting the information given in the instruction manual or normal wear and tear. In addition, the manufacturer is not responsible for damage caused by any parts of the appliance breaking or cracking.
- Must not immerse the appliance in water or any other liquid! If humidity gets into the appliance, it can cause defects, so do not use it in the bathroom or any other room where there is high humidity.
- Attention! Should the appliance, despite your caution, fall into water or make contact with humidity in any other way, please take it to the authorised service station and have it checked by an expert before re-using it! We have to call your attention to the fact that if the product becomes defective due to environmental effects like this, it will be considered as improper handling, and the manufacturer's warranty will lapse!
- If you do not use the appliance for a longer period of time (more than 1 month) then please remove the batteries from the battery holder. If you do not do this, corrosive chemicals may escape from the batteries and damage the appliance.

- The manufacturer excludes its liability for broken device parts or for any damage resulting from modifications, improper handling or cleaning, chemicals released from the device, incorrect battery replacement or from not correctly following the user's manual.
- Attention! The appliance must not be disassembled, its casing must not be removed, or its repair attempted. There are no parts inside the appliance that could be home repaired or used for any other purpose. In the case of any defect of the appliance please have it repaired only by an authorised special service station. The manufacturer does not undertake any warranty for any improper attempt at repair, or any intervention performed by unauthorised persons.

Keep your environment clean! The device cannot be disposed of with general waste, as it causes pollution to wildlife and severely endangers human health. At the end of its life please send in the product to an approved local recycling and environmentally friendly disposal facility for electrical devices. Do not dispose of flat batteries with general waste. Flat batteries are hazardous and pollute the environment if not disposed of in an environmentally friendly manner.

- The appliance can be energised only by batteries. Neither a mains connection, supply adapter nor supply accumulator can be used for its power supply.
- Please use only batteries as specified in the instruction manual. Other batteries (other size or shape) cannot be inserted in the appliance and it may become inoperable or damaged when used with batteries supplying a different voltage.
- Please always replace all the batteries at the same time and by the same type.
- Never dispose of the replaced batteries in the household waste, because they will endanger the environment! Please use a dedicated waste container provided for this purpose or a collecting station for disposing of used batteries!
- Never throw the batteries in the fire, because they can explode.
- Always switch the appliance off after use, and in case of longer storage remove the batteries from the battery holder.
- Take particular care of the blood pressure meter cuff and make sure it doesn't bend, stretch or wrinkle
- The appliance is intended only for adults and for human use, and it can only be used as specified in the instruction manual.

Important information!

We have to advise you that although the appliance outputs very precise measured values, the values can only be considered as informative data by which you can check the condition of your health, or in the case of persons under medical treatment, it can be used for regular self-checking of the blood pressure at home. The pressure gauge will provide precise values only if used as specified in the user's manual.

Should you happen to measure a blood pressure differing from the normal, healthy pressure, or if you have any doubt in respect of the measured value, please have your blood pressure checked by a specialist doctor for diagnostic purposes, so that you can control the value measured by the appliance. We recommend that you take the appliance with you and ask for the advice of the doctor.

In order to maintain permanent reliable and accurate measurement please, have the measurement accuracy of the device checked at an authorised measurement point, assigned for calibration, not later than after one year of use. The checking should be repeated at least yearly, for your safety.

- Do not use the appliance near to appliances emitting electromagnetic waves (e.g. mobile phones or microwave ovens). They can disturb the appliance resulting in faulty measurement and incorrect values.
- If after switching on nothing appears on the display, check whether the batteries have been correctly inserted and that they are correctly touching the battery contacts in the battery compartment.
- Please keep the appliance in its box when it is not in use, this way you can protect it against possible damage during storage.

Please keep this instruction manual. If you give the appliance to anyone else, you must give them this instruction manual, too!


MAIN COMPONENTS

1. Velcro cuff
2. LCD-display
3. On-/Off switch, "ON/OFF" button
4. Memory recall button "START"
5. Battery compartment
6. Battery compartment cover


BLOOD PRESSURE - GENERAL INFORMATION

Blood circulation ensures an appropriate supply of oxygen to the body. Blood pressure is the pressure the blood exerts on the artery wall. The systolic value (upper value) can be measured when the myocardium contracts and forces blood into the arteries. The diastolic value (lower value) is measured when the myocardium relaxes and blood flows from the veins into the heart.


Systolic blood pressure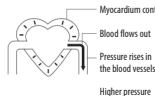


Diastolic blood pressure

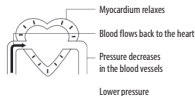In the human body, blood pressure increases naturally with age. This is the result of aging of the blood vessels. Increased blood pressure can be further aggravated by smoking, regular consumption of alcohol and drinks containing caffeine, extreme salt consumption, stress and lack of regular exercise. Also latent diseases such as problems with the kidneys or a high level of cholesterol (LDL) can cause increased blood pressure, because they result in the blood vessels losing flexibility. High blood pressure increases the risk of a stroke or myocardial infarction. Due to the fact that high blood pressure causes symptoms that can remain unnoticed for a long time, it is necessary to check our blood pressure regularly, to make sure that it is within the healthy range. A person's blood pressure fluctuates during the day, as a result of exercise or changes in our mood, and this is why only the average of several measurements or the number of changes can give us meaningful results.

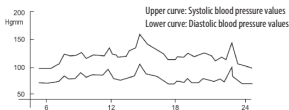The average of the measured blood pressure values can be evaluated on the basis of the figure below. Systolic values below 90 indicate low blood pressure.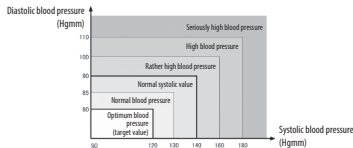


If the average of the blood pressure values measured by you differs from the normal values given in the table, or the measured values show significant fluctuations and thus they frequently differ from the healthy values, we advise you to consult your doctor. Only specialist doctors can make a correct diagnosis, based on measurements made with medical instruments. Furthermore, we recommend that you take your blood pressure monitor with you, as it is necessary to check the calibration of your device from time to time.

In addition to the medical treatment prescribed by your doctor you can promote efficient reduction of your blood pressure by observing the following suggestions:

- Slimming course or diet that would reduce the level of harmful cholesterol (LDL)
- Reducing the consumption of drinks containing alcohol and caffeine
- Giving up smoking
- Moderate salt consumption
- Regular exercise

POWER SUPPLY

The device operates on 2 x 1.5 V AAA batteries.

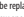1. Slide the battery compartment cover as shown by the arrow, then remove it.
 2. Insert the batteries in the battery holder according to the marked polarity.
 3. Finally replace the battery compartment cover, making sure it is securely fitted.
- Batteries must be replaced when the battery icon  appears on the display. Both batteries must be replaced by a new ones. You must use alkali batteries only. Ni-Cd/Ni-MH batteries have an output voltage not exceeding 1.2V, which is insufficient to operate the blood pressure measuring instrument.

- If you do not want to use the device for a long period of time, remove batteries from the battery compartment as they can leak - caustic electrolyte spilled out may damage the device (in which case the manufacturer's warranty becomes void). In order to avoid damage, you must follow these instructions:
- Batteries must be put into the compartment with their correct polarity (+ and -), as marked on the battery compartment.
- Never use new and used batteries together.
- Never use batteries of different manufacturer and type together.
- Used batteries must immediately be replaced by new ones. If you do not want to use the device for a long time, remove batteries from the battery compartment.

USING OF APPLIANCE

Blood pressure measurement:

In order to obtain reliable measurements, do not do any strenuous physical activities before beginning measurements. Rest for about 5 to 10 minutes before taking measurements. Just before or during measurements do not eat, drink or smoke. Certain irregularities, e.g. changes in heartbeat, fibrillations might cause erroneous measurement results.

1. Free your forearm by removing any clothes obstructing your arm's skin, as the blood pressure meter cuff needs to touch the skin surface. Do not wear watches, bracelets etc. during measurement.

2. Rest your elbow on a table.

3. Open the velcro of the blood pressure meter and pass your hand through. Tighten the device by using velcro at a distance of about 1 cm from your hand as shown in the figure.

4. Turn your palm upwards, hold fingers and wrist in natural position during measurement. Do not double your fist or let your wrist hang down. During the measurement hold your forearm with your other hand as shown in the figure.

Blood pressure measurement process

1. Push down the "ON/OFF" button. All the symbols will appear on the display for a short time. This lets you check the fail safe operating status of the display. After self-test, the sign "0" flashes a few times then the meter begins pumping up the cuff.
2. The apparatus pumps up the cuff automatically to a magnitude of 190 Hg mm, then it decreases the pressure gradually. In case of a higher blood pressure value the apparatus automatically pumps the cuff further twice or three times for the sake of a successful measurement. During the measurement the symbol of pulse marked with a heart blinks following the rhythm of the pulse.

3. At the end of the measurement a sound signal can be heard and the blood pressure and the pulse values appear on the display. The measured values are automatically stored within the memory of the apparatus, as well.
4. After measurements have been taken, the device can be shut down by pressing the "ON/OFF" button or will automatically be switched off after 3 minutes if left unused.
Note: Wait about 5 to 10 minutes before starting a new measurement. This time is necessary for vessels to recover their original status after measurement. Taking into consideration psychologic factors, for some people a longer delay could be justified between two measurements.

Memory function:

After measuring, both blood pressure and pulse data will automatically be stored in the device's memory. The memory is able to store the last 120 measurements. When the memory is full the latest measurement data will replace the earlier ones.



1. The memory function of the apparatus can be activated after the measurement, or with the apparatus switched off. With pressing the START button once after the measurement, the results of the earlier measurements can be displayed.
2. With pressing the "START" button again the data measured before can be displayed in a reverse chronological order. You can see the symbol "No" and the number of the stored value.

To delete the data stored within the memory, first switch on the turned-off apparatus with pressing the "START" button once, then press and hold the "ON/OFF" button until horizontal lines appear on the screen in the place of the numbers.

DEVICE MAINTENANCE AND CLEANING

- When not in use, store the device in its plastic box in a dry, dust- and frost-free place. Do not expose to direct sunlight, or to extreme external temperatures.
- If you do not want to use the blood pressure meter for an extended time, remove batteries from the battery compartment.
- Clean both the cover and cuff of the device with a moist fluff-free cloth, then wipe it thoroughly dry with a dry cloth.
- Completely dry the device before using or storing it.
- In order to obtain accurate and precise measurements, check the device yearly with your doctor.
- During cleaning, never submerge the device or its cuff into water or hold under a water tap.
- Do not dismantle the device.
- When cleaning the appliance, never use solvent (acetone, benzole, petrol, alcohol, etc.) as these may solve the casing material, or the internal parts of the appliance if they get into. Do not use cleaning agents with coarse or grinding effect.

Troubleshooting
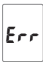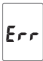
| Failure | Possible cause | Troubleshooting |
|--|--|--|
|  | Problems detected during pumping up | Check cuff position then repeat measurement |
|  | Measurement error detected | Do not move nor speak during measurement |
|  | Low battery | Remove batteries and replace with new ones |
| Batteries are correctly installed but nothing or error is displayed when switched on | Batteries are not in the right position within the battery compartment or are flat | Check polarity of batteries or replace with new ones |

Measurement method:

Oscillometric

Measurement range:

Blood pressure domain: 0 to 300 Hgmm

Measurement accuracy:

Pulse frequency: 30 to 180/minute

Blood pressure: +/- 3 Hgmm

Pulse frequency: +/- 5% of value measured

Memory capacity:

120 data measured

Pumping up:

Automatic, with internal pump

Pressure release:

by an automatic release valve

Weight:

about 100 g (without batteries)

Cuff wrapping:

13.5 to 21.5 cm.

Manufacturer:

Hangzhou Sejoy Electronics & Instruments Co., Ltd.;

Building 2, No. 202, Zhenzhong Road, West Lake Economy &

Technology Zone, 310030 Hangzhou, China

Type number:

BP201

Authorized EU representative:

ShangHai International Holding Corp. GmbH (Europe)

Eiffenstrasse 80, 20537 Hamburg Germany

Importer:

dyras www.trading Kft. 1108 Budapest, Gyömrői str. 140.

Hungary

Trademark owner:

The dyras www.factory LLC 444 Brickell Avenue / Suite S1103

Miami, FL 33131 U.S.A.

We are continuously developing our products to keep them up-to-date, which means the technical data of our appliances may change without any prior notice. Errors and omissions excepted.

QUALITY CERTIFICATION

It is hereby certified that the wrist blood pressure meter type dyras BPSS-4129 conforms to the technical specification described below:

| | |
|---------------------------------------|----------------------------------|
| Type of device: | BPSS-4129 |
| Description: | Wrist type blood pressure meter |
| Nominal operating voltage: | DC 3 V (2 x 1.5 V battery "AAA") |
| Rated operating temperature: | +5° C to +40° C |
| Rated operating moisture content: | ≤85% R.H. |
| Rated operating atmospheric pressure: | 106 to 70 kPa |
| Rated storage temperature: | -10° C - +55° C |
| Rated storage moisture content: | <95% R.H. |
| Display: | LCD digital display |
| Measurement location: | wrist |

COPYRIGHT

© Copyright The dyras www.factory LLC U.S.A.

All rights - including distribution by film, radio or television, translation, photographic or audio copying, abstract reprint and other duplication - are specifically reserved. Any breach may result in legal action.

THE SYMBOL CE

This appliance meets the requirements for safety and electromagnetic compatibility (EMC). The relevant 'CE' mark can be found on the nameplate, on the packing box and in the instruction manual.

Dear Customer,

Please visit our website at www.dyras.com where you can learn more about your appliance, new dyras products and our activities.

You can also register your product online by completing the registration form on our homepage. Registration offers a number of advantages:

- You'll receive the latest information, news and recommendations about your new product, either by email or by post.
- We will inform you about any changes to our service network and the availability of accessories and spare parts.
- We'll let you know if any new parts or accessories are available to improve your product (and, for electronic equipment, if a software/firmware upgrade is available to enhance its functions).
- You'll be the first to hear about any innovative, new products we release.

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© Copyright The dyras wwe.factory LLC U.S.A.
© Copyright The dyras ww.factory LLC U.S.A.